

ARROW™

Arrow™ EZ-IO™

Quick User Guide
Pediatric Application



Teleflex™
Empowering the future of healthcare




1. Identify

Identify the insertion site



2. Landmark

Landmark of the insertion site

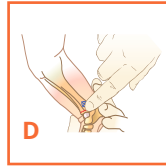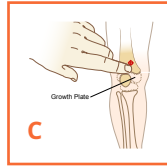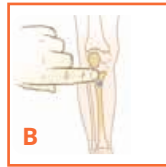


- A. Proximal Humerus
- B. Proximal Tibia
- C. Distal Femur
- D. Distal Tibia


3. Assess

Assess skin Thickness


Examine the insertion site to approximate tissue depth

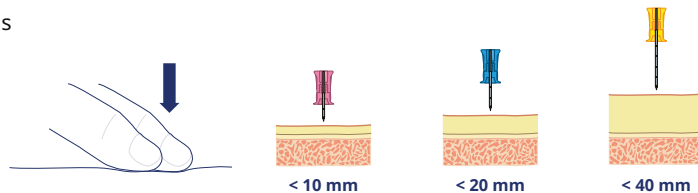



4. Confirm and Insert EZ-IO™ Needle

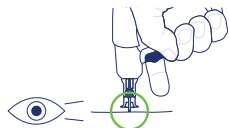

Insert the needle until the tip rests against the bone and confirm that at least one black line is visible (minimum 5 mm)

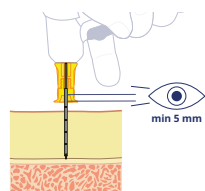


PROCEED

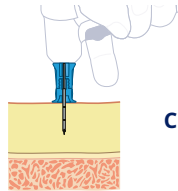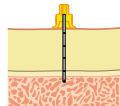


If at least one black line is not visible, remove the needle and move to the next size up

CHANGE

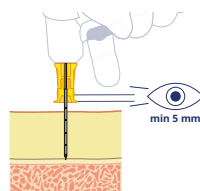


PROCEED

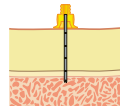


5. Engage the EZ-IO™ Driver

Squeeze the trigger, apply gentle steady pressure.

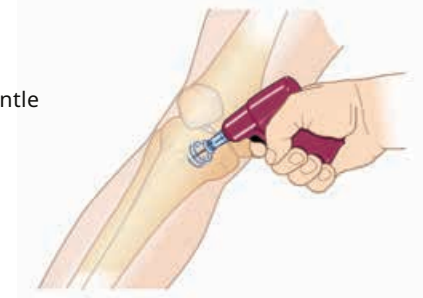

6. Secure

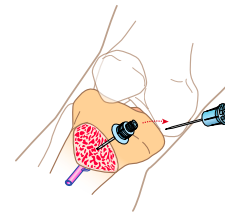

Stabilise hub and disconnect driver and remove stylet.



Place stylet into red NeedleVISE for sharps containment.



Apply EZ-Stabilizer Dressing over the catheter hub.




Attach primed EZ-Connect™ Extension and EZ-Stabilizer dressing by pulling the tabs to expose the adhesive and adhere to the skin.

7. Deliver

Set and rapid flush saline (2-5 ml for infant/child)



Administer fluids and medication. For optimal flow, infuse with pressure.

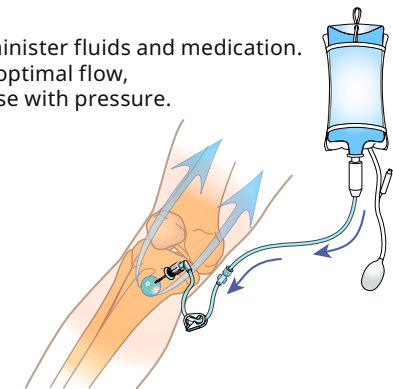

About Teleflex

As a global provider of medical technologies, Teleflex is driven by our purpose to improve the health and quality of people's lives. Through our vision to become the most trusted partner in healthcare, we offer a diverse portfolio with solutions in the therapy areas of anesthesia, emergency medicine, interventional cardiology and radiology, surgical, vascular access, and urology. We believe that the potential of great people, purpose driven innovation, and world-class products can shape the future direction of healthcare.

Teleflex is the home of Arrow™, Barrigel™, Deknatel™, QuikClot™, LMA™, Pilling™, Rüsch™, UroLift™ and Weck™ – trusted brands united by a common sense of purpose.

At Teleflex, we are empowering the future of healthcare. For more information, please visit teleflex.com.

Corporate Office

Phone +1 610 225 6800, 550 E. Swedesford Road, Suite 400, Wayne, PA 19087, USA

Regional Offices

United States: Phone +1 919 544 8000, Toll Free 866 246 6990, cs@teleflex.com,
3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

Latin America: Phone +1 919 433 4999, la.cs@teleflex.com,
3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

International: Phone +353 (0)9 06 46 08 00, orders.intl@teleflex.com, Teleflex Medical Europe Ltd.,
IDA Business and Technology Park, Dublin Road, Athlone, Co Westmeath, Ireland

Teleflex, the Teleflex logo, Arrow and Arrow EZ-IO are trademarks or registered trademarks of Teleflex Incorporated or its affiliates, in the U.S. and/or other countries.

Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative. Revised: 09/2024.

© 2024 Teleflex Incorporated. All rights reserved.

MCI-2024-1044-EN · REV 0 · 09 24 PDF